



Bio-Therapeutic

Skin Care for life inside and out[®]

bt-micro[®]

How-To Guide: Overlay Replacement

Supplies Needed

X-acto knife or similar blade
Opti-Cide wipes

Parts Needed

MDX-129 bt-micro overlay




Bio-Therapeutic
Skin Care for life inside and out®





Removing overlay

Lay the blade flat against the bt-micro. Gently and carefully separate the overlay from the plastic case by sliding the blade under the overlay at one corner.

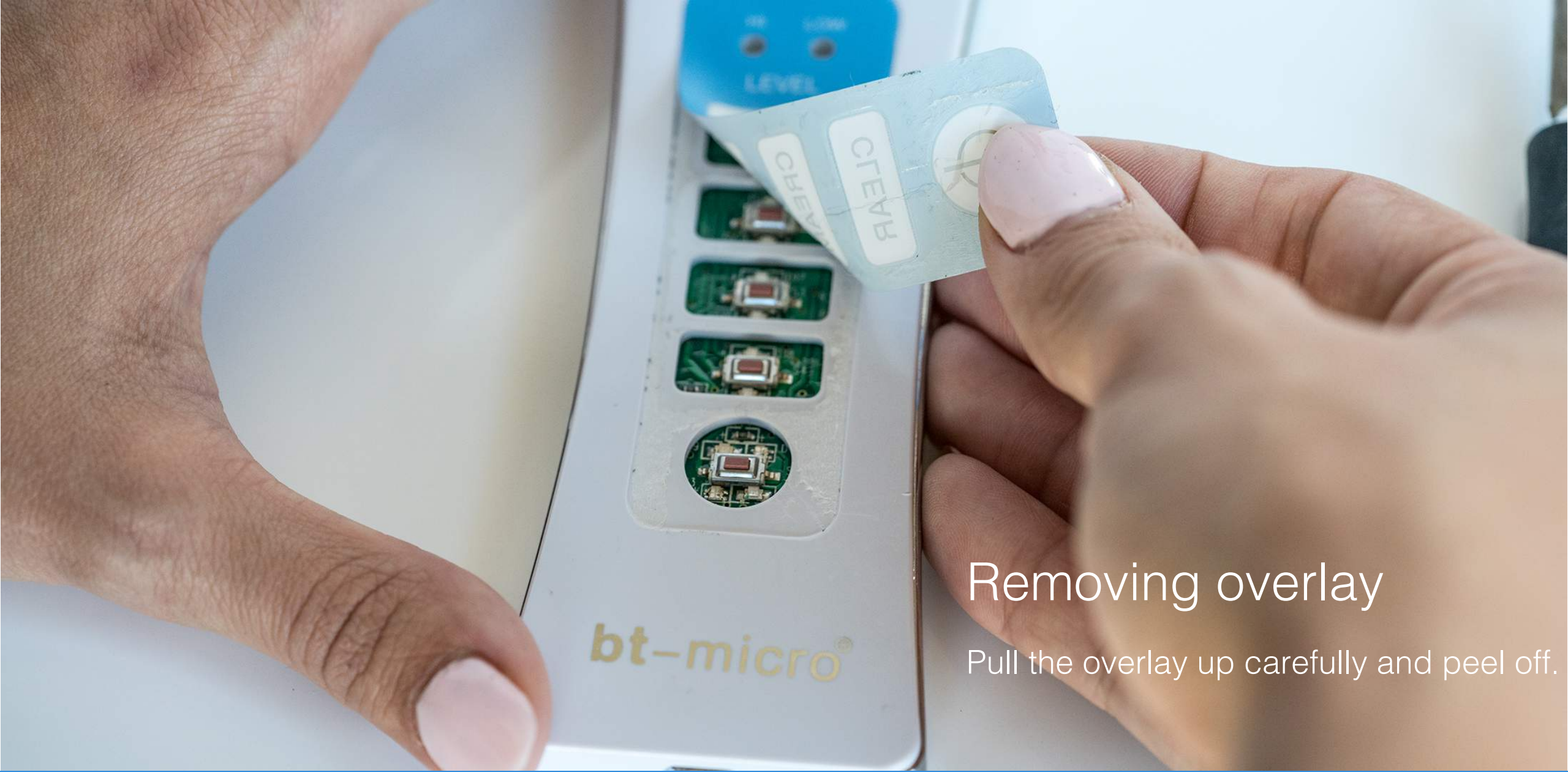

Removing overlay

Pull the overlay up carefully and peel off.

bt-micro®

Prepping the surface

Clean off any debris or adhesive that remains with an Opti-Cide wipe.






Installing new overlay

Remove the backing from the bt-micro overlay to expose the adhesive.



Bio-Therapeutic
Skin Care for life inside and out®



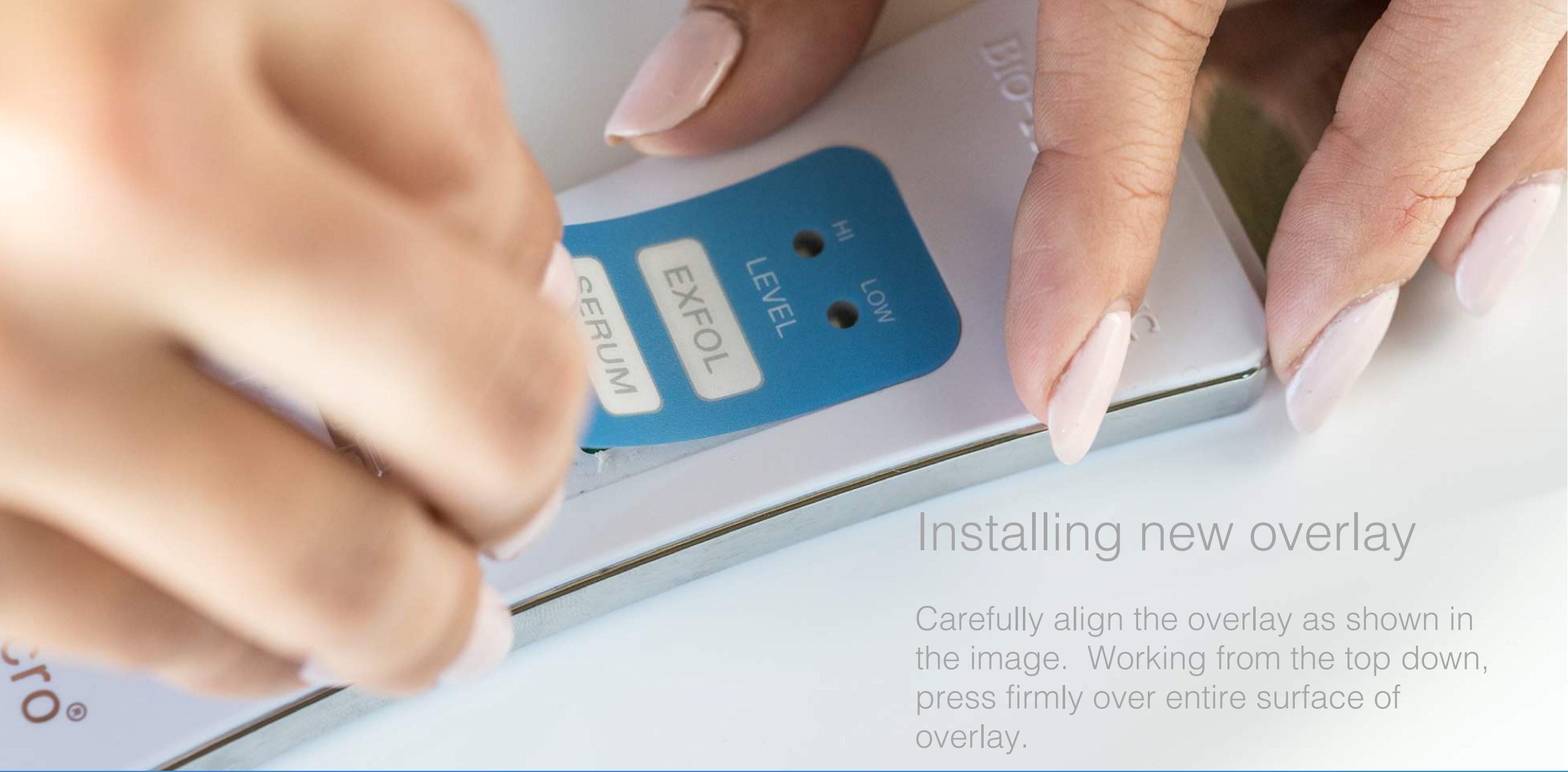

Installing new overlay

Carefully align the overlay as shown in the image. Working from the top down, press firmly over entire surface of overlay.





Installing new overlay

Working from the top down, press firmly over entire surface of overlay to secure.



Bio-Therapeutic
Skin Care for life inside and out®

